Premium J6

Light Treatment System

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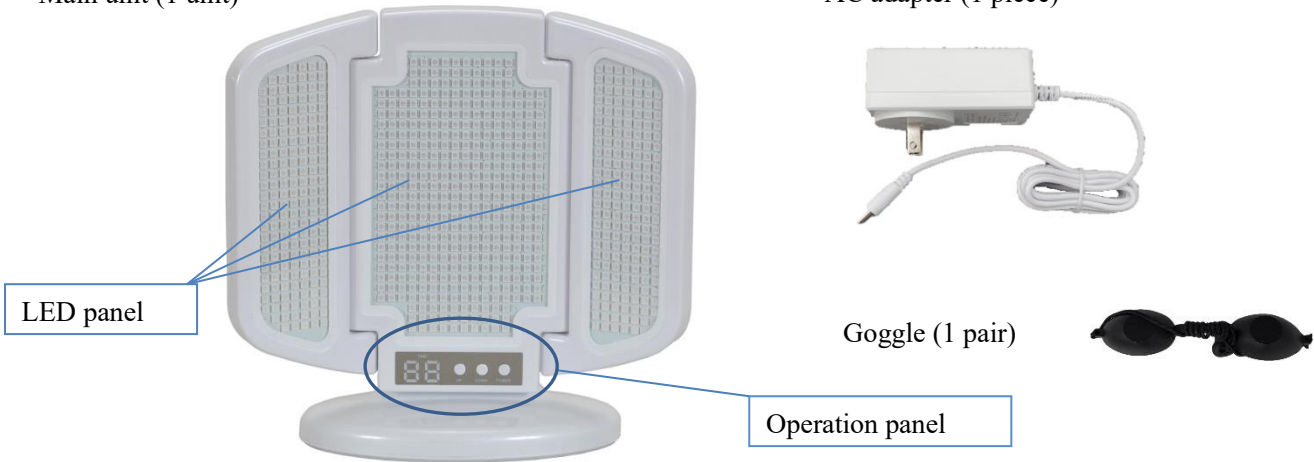
Contents and parts identification

Main unit (1 unit)

AC adapter (1 piece)

Operating instructions (including warranty card) (1 copy)

How to use

1. Insert the AC adapter into the connection port on the back of the main unit. Press POWER on the operation panel once to turn on.
TIME display shows 15. Use the UP/DOWN buttons to adjust the time.
You can set 1 to 30 minutes for TIME.
2. When you press POWER again, a red LED turns on. When you press POWER again, an orange LED turns on. The LED changes every time POWER is pressed. (Red --> orange --> yellow --> blue --> indigo --> purple --> off)
3. Wear the goggles and irradiate.
If you feel your skin is dry during irradiation, please replace the fluid (SO₂S recommended) during use.
4. LED turns off with a beep sound When TIME ends, which means POWER OFF.

[Operation panel]

TIME

You can set 1 to 30 minutes.
Press the UP/DOWN buttons to adjust.
You can operate during treatment.

POWER button

Press once to turn on the unit, then TIME display shows 15. When you press POWER again, a red LED turns on. The color of LED changes every time POWER is pressed. The power is turned off at the end.

Features of the device

This is a LED beauty device especially for beauty.

A total of 746 LEDs, red, orange, yellow, blue, indigo and purple, are used

Irradiation time can be set from 1 to 30 minutes.

Red LED wavelength: 620 nm±10

For people who are worried about dry skin

Maintain firmness and gloss of skin

Blue LED wavelength: 480 nm±10

For people who are worried about rough skin including p

Control sebum secretion and acne

Sedate your skin

Yellow LED wavelength: 580 nm±10

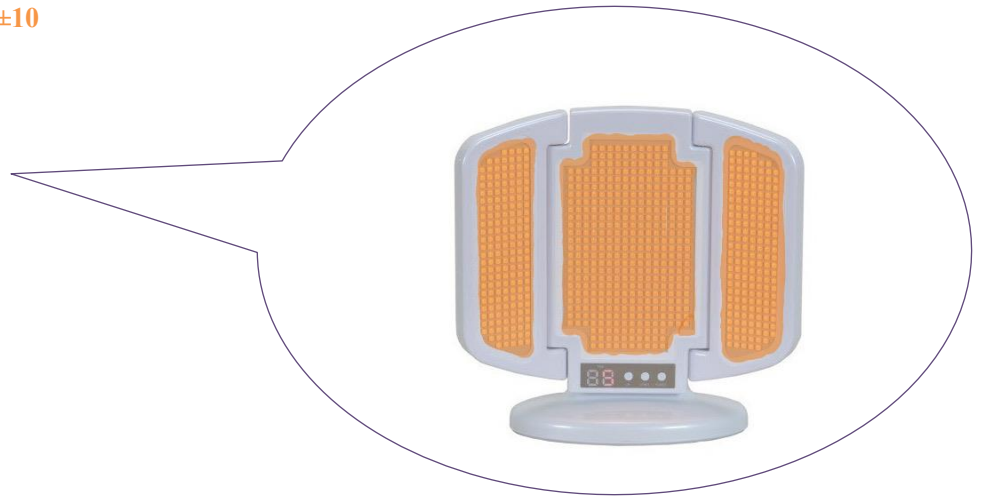
For people who feel their skin is in bad condition

Relax and sedate sensitive skin


Orange LED wavelength: 600 nm±10

Maintain healthy skin
Vitalize your skin mildly


Indigo LED wavelength: 445 nm±10

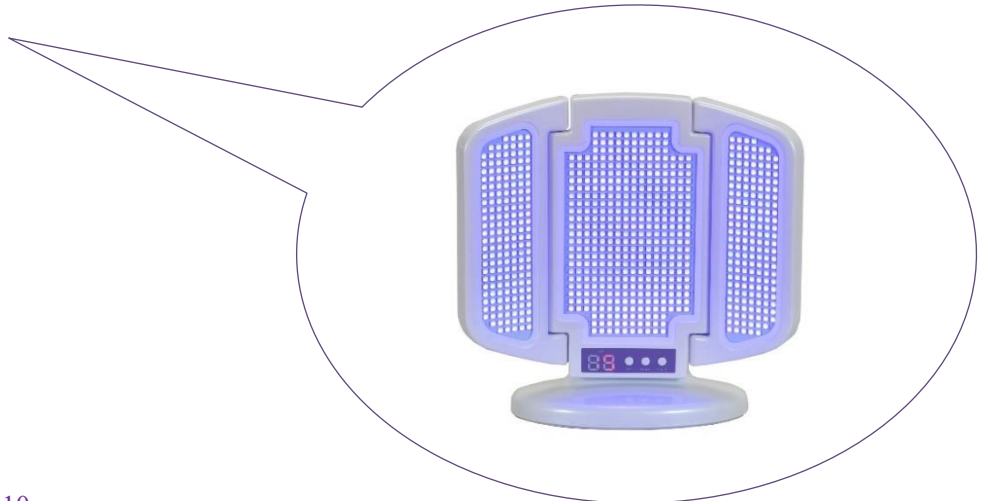For sun damaged skin

Purple LED wavelength: 405 nm±10

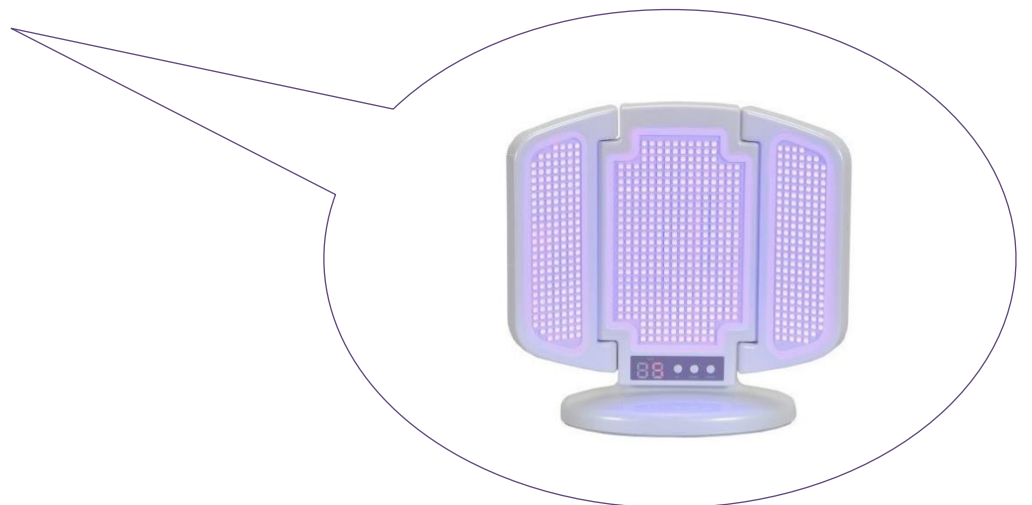For people who are worried about sagging skin
Relieve redness in the face

- * This device is not a medical device.
- * There are differences in the effectiveness on the skin among individuals.

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Specifications

Product name	: Premium J6
Power supply	: AC 100 V 50 Hz/60 Hz
Power consumption	: 40 W
Illuminant	: Red LED wavelength 620 nm±10 Orange LED wavelength 600 nm±10 Yellow LED wavelength 580 nm±10 Blue LED wavelength 480 nm±10 Indigo LED wavelength 445 nm±10 Purple LED wavelength 405 nm±10

Size: Width 320 mm, depth 130 mm, height 330 mm

Weight: 1.5 kg

* The specifications are subject to change without notice because of improvements.

Service

The device comes with a quality warranty for one year after purchase. This warranty only covers technical defects and failures. Also, this warranty does not cover failures which occurred because you did not follow the operation and installation instructions described in this warranty even within the warranty period. This warranty also does not cover the following failures.

- Malfunctions or failures caused by inaccurate operation.
- Failures due to use in an unsuitable environment, or malfunctions and failures caused by dismantlement, falling, collision, etc.
- Malfunctions and failures due to natural disasters.
- Malfunctions or failures caused by inferior environment including humidity or dust.
- Malfunctions and failures due to modification.

If you have any questions about services, please contact the following.

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